

Instructions – Propolis Vaporizer & Inhaler A4



Usage of the Breathing Mask

The propolis vaporiser "A4" has its own programme especially for the application "inhalation" with the breathing mask. The switch button for this has the symbol "P" (inhalation mode, breathing mask, inhalation). In order to successfully inhale the propolis vapours, the capsule in the device must first be hot enough to release the essential oil vapours. Then it must be ensured that these propolis vapours fill the entire tube well. Therefore, before inhaling, the device must be operated for at least ½ hour, preferably one hour, on programme "P" with hose connection and mask. As soon as the essential oils (76 detected so far) reach the breathing mask, i.e. the tube is well flooded, the application can begin.

If you use the programme with the "ventilator" symbol, you will receive propolis vapours for the room air at intervals. This is too little for inhalation. With the "P" symbol, the fan runs continuously and the capsule is heated to a slightly higher temperature, as the inhalation and exhalation cools the capsule so much that no more vapours could be secreted.

There are 2 breathing masks in the range. One for children and one for adults. If you wish, we can also supply German PARI masks in addition to the original green masks. To be able to put these masks on the tube, you need a transition piece. We can also supply this. Please contact us. Additional masks and tubes can be ordered at any time.


IMPORTANT NOTICE

By ordering the goods and paying the invoice, the customer declares that he is fully responsible for the use of all inhalation attachments, as well as for the propolis vaporiser A4.

The seller as well as the manufacturer do not assume any responsibility.

This is a product for room hygiene.

The reseller is also obliged to inform his customers of this in writing.

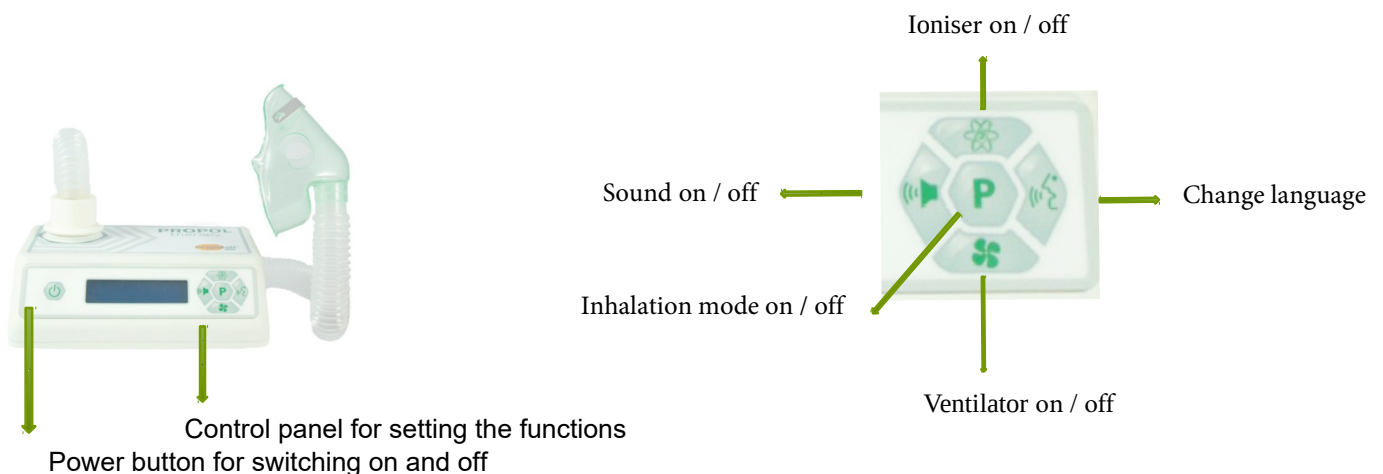

Display: The display shows which functions are switched on or off.

I = Ioniser ; V = Ventilator ; S = Sound ; D = Remaining running time of the capsule

The display switches off after 30 min and can be switched on again using one of the right-hand buttons.

Restarting of the device: Press the power button for 8-10 seconds, then release it.